



English

Baby Electric Waterproof Nasal Aspirator

Easier breath Healthier growth

Chinese ➤ English

PRODUCTS DESCRIPTION

This product is an electric waterproof nasal aspirator(X 11 waterproof), which is used for cleaning nasal secretions of babies. The product has 4 different silicone tips, which are the gourd-shaped (flat, gap) suction tip and funnel-shaped (flat, oblique) suction tip respectively, suitable for babies of different ages. These tips are made of silica gel, which is soft, non-irritating, safe and environmentally friendly. Different shapes of silicone tips make it easy to absorb secretions from different parts. Large-capacity lithium battery and USB charging design make it more convenient to use and carry.



This operation manual contains safety attentions and instructions for the proper use of the product. Therefore, please read the manual carefully before use and keep it properly so as to avoid loss and hurt caused by improper use.

PRODUCT FEATURES

- **Replaceable Design Of The Silicone Tip**

There are 4 different suction tips, which can be flexibly chosen according to baby's age and needs.

- **Adjustable Operation Mode**

Soft, moderate and strong suction mode, can be switched freely according to your own needs, and designed with 3 minutes automatic power-off function, to prevent machine idling for a long time.

- **Easy To Clean**

This product is six grade waterproof, which can absorb water and wash water (not immerse in water). The suction head and storage bin components can be removed for cleaning, which can better clean the residue and clean the machine more thoroughly.

- **Easy To Carry**

Rechargeable design, compact, light weight, easy to carry.

- **Physical Nursing**

The silicone suction head for babies is very soft, non irritating, safe and hygienic. Different shapes of suction heads are designed for babies of different ages. Better protect your baby's nose.

- **Large Capacity & Rechargeable**

Equipped with built-in 500mAh large capacity rechargeable lithium battery and USB portable charging port design, the machine can be charged via computers, power banks and any other devices with a USB port. You can use it at any time whether at home or out.

- **Night Mode**

The LED lamp can be turned on when the device is in operation. The warm color light will not irritate the baby's skin and eyes. It is convenient to use at night and helps to see the inside of the baby's nasal cavity.



GENERAL INTRODUCTION

Reservoir (Removable) Reservoir (Removable)

Silicone Tip

Led Light

USB Charge Port



Outfall

Suction Mode Indicator
(Strong, Moderate, Soft)

Mode & Power Button

Led Light Button

COMPONENTS INFORMATION



Gourd-shaped silicone tip

Suitable for newborn babies



Gourd-shaped Gap tip

(It is more convenient for nasal discharge)

Suitable for newborn babies



Funnel-shaped tip

Suitable for babies over one-year-old



Funnel-shaped oblique tip



Reservoir (Removable)

Anti-reflux cap
of reservoir

reservoir

The outer cover
of reservoir

Rear storage bin cover

INSTALLATION

1. Cover the reservoir with anti-reflux cap



2. Install the assembled reservoir into the groove of the aspirator body



3. Then install the outer cover on the body and then tighten



4. Choose the suitable tip and install it on the outer cover clockwise to ensure the suction part is working properly of reservoir



INSTALLATION ORDER

When installing, please pay attention to the installation sequence of components and please tighten it clockwise to ensure that the parts will not fall off.

Reservoir -> Anti-reflux cap of reservoir ->

The outer cover of Reservoir -> The silicone tip



Note

When you see that the liquid in the rear storage bin reaches half, you need to remove and clean the front and rear bins of the product, otherwise the spot will overflow outside the fuselage. After the cleaning is completed, the machine is installed and the machine is turned on to the maximum position and it is idle to automatically shut down to remove the moisture inside the machine. And then collect and place



SPECIFICATION

Product Name	Electric Baby Nasal Aspirator
Model No.	X11
Charging Voltage	5V/==
Operating Voltage	3.7V
Power	2.2W
Battery Capacity	500MAH
Executive Standard	GB4706.1 GB4705.15

INTRODUCTIONS OF SILICONE TIPS



Gourd-shaped Tips (Flat, Gap)

Suitable for newborn babies, smaller tips can be appropriately extended into the nostril. The hoist can block off nostrils, making it easier to suck out the nasal mucus. The gap tip can be better close to the sidewall of the nasal cavity, and it will be easier to absorb attached nasal mucus.



Funnel-shaped Tips (Flat, Oblique)

Suitable for children over 1 year old. The funnel-shaped tip goes deeper into the nasal cavity, which is more suitable for deeper nasal mucus. The oblique tip can be better close to the sidewall of the nasal cavity, and it will be easier to absorb attached nasal mucus.

HOW TO USE

Firstly assemble the nasal aspirator according to the section "Assembling". Please make sure that the seal ring of the aspirator body has been fixed and install a suitable silicone tip on the outer cover. At last, choose the appropriate mode according to your needs.

Specific Method

1. If the baby's snot is dry, it is better to soften the mucus with saline first and then use the electric nasal aspirator. This will achieve a better effect.
2. If the baby's mucus is thin, you can directly use the electric suction device to suck out.
3. If you can not see the baby's mucus but can hear the sound of "wheeze", the baby is also in a state of nasal congestion. It is recommended to gently stimulate the baby's nose and let the baby sneeze and then use the electric nasal aspirator suck out.

* Before using the electric nasal aspirator for the baby, parents should first test the suction with the back of the hand and adjust it to the appropriate mode.

Use Suggestions

For everybody's interest and to participate actively in protecting the environment, please do not stack together with other normal household waste when throwing away the product. It should be handed over to an official designated recycling center that will be better for environmental protection.

The battery must be removed from the appliance before the device is discarded. When removing the battery, the appliance must be powered off. The battery should be disposed of safely.

Use Suggestions

-  Please charge the battery for 2 hours before using it for the first time to extend battery life.
-  Press the "button" once for soft suction mode
Press the "button" twice for moderate suction mode
Press the "button" three times for strong suction mode
Then press "button" again for shutdown mode

CLEAN AND MAINTENANCE

1. Before use, please make sure that the working chamber components have been cleaned and there is no liquid residue, so as to avoid bacteria breeding and affecting suction.
2. Please do not let children use this product by themselves. Please adjust the suction on the back of adult's hand before use.
3. During cleaning and maintenance, please turn off the machine and do not immerse the machine body in water. The suction head and storage bin can be removed for separate cleaning. The working warehouse can be washed with a faucet. A small amount of medical alcohol can be dipped into a cotton swab to clean the air inlet. After cleaning, make sure that the parts are completely dried before assembling, so as to prevent the breeding of bacteria (do not use sharp, corrosive and volatile cleaning substances, so as to avoid affecting the service life of the machine).
4. Please use the matching USB cable for charging, and ensure that the charging voltage meets the product requirements.
5. Do not charge during use.
6. If it is not used for a long time (about three months), please take out the machine and fully charge it before storage.
7. Please do not disassemble or repair the product by yourself. For maintenance, please contact the shop where you purchased the product.

