Clean Bum Menstrual Cup Sterilizer

Handleiding User Manual Manuel d'instructions

Clean Bum
Akatrade VOF 2518 TN 193, the Netherlands
www.cleanbum.eu www.cleanbum.nl

Handleiding

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User Manual

Thank you for purchasing the Clean Bum menstrual cup sterilizer. Please read the instruction manual for further reference before using the machine for the first time.

Safety precautions:

- To avoid burns on the body caused by boiling water and steam, make sure to close the lid when using the sterilizer, children should stay away from the product.
- Do not approach the steam vents when it is functioning to avoid burns.
- Please don't place the non heat-resistant items in the menstrual cup sterilizer to prevent damage to the items.
- When cleaning the product, please cut off the power, wipe with a clean soft cloth or alcohol, and avoid using organic solvents such as gasoline.
- Non-professionals do not disassemble or repair the product, in case of causing damage to the product.
- After using the sterilizer, unplug the power cord and store it in a dry place.

Product specifications:

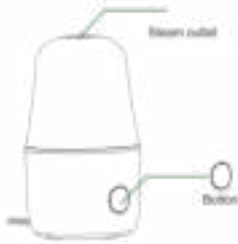
Product Name: Clean Bum Menstrual Cup Sterilizer, model Clean Cup green (MCS2E)

Input: 220V

Rated power: 80W

Water line sterilization time: 8~10 minutes

Power indicator working status: light.

Features:

- This product is specially designed for menstrual cups made of silicone, rubber or TPE.
- Large water capacity, long-term high-temperature disinfection, melting and cleaning all kinds of dirt, effectively killing bacteria up to 99.5%.
- Environmentally friendly material Polypropylene (PP), high temperature resistant, corrosion resistant, double insulated design to avoid burns.
- Built-in fuse to guarantee electric safety. Heating sheet is coated with Teflon, which is durable.
- One-button operation, antiburn material.
- Stylish and beautiful appearance, convenient for travel. Musthave for woman.

How to use:

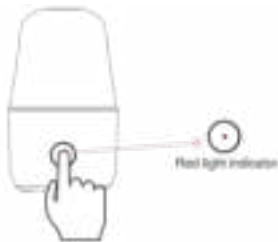
1

Fill up the small circle with water until the top.

2

Put the menstrual cup into the disinfection chamber and close the top cover.

3

Plug the power cord of the Clean Bum into the outlet. Press the power button, the indicator light lights up, indicating that the machine start working.

4

The sterilization time will be about 8~10 minutes. There is anti-dry function, the power will be automatically cut off when the water is dried.

5

Let it cool before take out and use the menstrual cup.

6

Important: please unplug the power cord when the disinfection is finished.

Tips

- Before disinfecting the menstrual cup, wash the menstrual cup and clean the sterilizer.
- The menstrual cup can be placed in the sterilizer and stored in a dry place.

Service

For product quality problems caused by non-human factors within one year from the date of purchase, please contact the dealer for complete machine replacement. Please provide a description or picture when the quality issues involved.

Non-warranty

- Incorrect use and human-induced damage.
- Cannot provide proof of purchase.

Decalcifying

After a while, limescale can be build up in the sterilizer. This is not dangerous to human health.

- Fill the Clean Bum with a mixture of water and vinegar.
- Turn on the sterilizer and let it boil for 8 minutes. Repeat if necessary.
- Repeat the process with clean water.
- Dry the sterilizer.

Questions? Contact us through info@cleanbum.nl

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