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The advertisement features a large, stylized blue tooth graphic on the left side. At the top right, the 'optimalwhite' logo is displayed in a blue, sans-serif font, with 'ADVANCED TEETH WHITENING' in a smaller font below it. In the center, the text 'optimalwhite' is written in a large, blue, sans-serif font, with 'TEETH WHITENING KIT' in a smaller font below it. To the right of this text are five blue stars. At the bottom of the advertisement, there are four icons: a green leaf labeled 'MINT', a clock icon labeled 'FAST RESULTS' with '10/20 min' inside, a green leaf labeled 'VEGAN', and the American flag labeled 'MADE IN USA'.

EN HOME WHITENING KIT

INSTRUCTIONS:

1. Brush and floss your teeth.
2. Use the shade guide to find your current teeth shade.
3. To activate the LED lamp, open the back of the device, remove the batteries and the transparent plastic. Re-insert the batteries.
4. Attach the mouth tray to the LED lamp.
5. Apply a thin line of gel to the mouth tray (upper and lower arch) and spread it. It requires less than 1 mL of whitening gel per arch.
6. Insert the mouth tray in the mouth and turn the LED lamp on by pressing the white button.
7. Wear the tray for 10 minutes. The LED lamp will automatically shut off after 10 minutes.
8. Remove the tray and rinse your mouth with water.
9. Use the shade guide to compare your results.
10. Separate the LED lamp from the mouth tray. Wash the tray with cold water and store it in a dry place.

Repeat steps once a day until desired shade is obtained.

WARNINGS: Not to be swallowed. Keep away from eyes. Keep out of reach of children. Minimum age is 18 years old. Do not use while sleeping. Do not use if you are pregnant / nursing, have poor dental health (decaying teeth, exposed roots, gum disease), wear braces or had recent oral surgery. Will not whiten caps, crowns, laminates, white fillings or veneers. Results vary from person to person.

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