# C CARE

## TH838

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**Fabrikant:** Radiant Innovation Inc.  
IF, No.3, Industrial East 9th Road,  
Science-Based Industrial Park, HsinChu Taiwan

**Geautoriseerde vertegenwoordiger in de Europese Unie:**  
Medical Technology Promedt, Consulting GmbH  
Altenhofstrasse 80, D-66386 St., Ingbert, Germany

**Importeur:** CRESTA INTERNATIONAL B.V.  
Rollbrigweg 4 - 1332 AS - Almere - The Netherlands  
Info@crista-international.com + 31 20 6 932 932

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# C CARE

## Appendix

<b>Manufacturer's declaration-electromagnetic emissions</b>		
The TH8xyz series is intended for use in the electromagnetic environment specified below. The customer or the user of the TH8xyz series should assure that it is used in such an environment.		
<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
RF emissions CISPR 11	Group 1	The TH8xyz series uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The TH8xyz series is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

<b>Manufacturer's declaration-electromagnetic immunity</b>			
The TH8xyz series is intended for use in the electromagnetic environment specified below. The customer or the user of the TH8xyz series should assure that it is used in such an environment.			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment - guidance</b>
Electrostatic discharge(ESD) IEC 61000-4-2	Contact: $\pm 8$ kV Air $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV	Contact: $\pm 8$ kV Air $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50, 60 Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The TH8xyz series power frequency magnetic fields should be at levels characteristic of a typical location in a typical home healthcare environment.

# C CARE

<b>Manufacturer's declaration-electromagnetic immunity</b>			
The TH8xyz series is intended for use in the electromagnetic environment specified below. The customer or the user of the TH8xyz series should assure that it is used in such an environment.			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment - guidance</b>
Radiated RF IEC 61000-4-3	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz	<p><b>Recommended separation distance</b></p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = 2.3 \sqrt{P} \text{ 800 MHz to 2.7 GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the TH8xyz series is used exceeds the applicable RF compliance level above, the TH8xyz series should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the TH8xyz series.			

# C CARE

## Recommended separation distances between portable and mobile RF communications equipment and the TH8xyz series

The TH8xyz series is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the can TH8xyz series help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the TH8xyz series as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3 \sqrt{P}$
0.01	N/A	0.12	0.23
0.1	N/A	0.38	0.73
1	N/A	1.2	2.3
10	N/A	3.8	7.3
100	N/A	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

# C CARE

<b>Manufacturer's declaration-electromagnetic immunity                      Test specifications for ENCLOSURE PORT IMMUNITY to RF                      wireless communications equipment</b>			
The TH8xyz series is intended for use in the electromagnetic environment specified below. The customer or the user of the TH8xyz series should assure that it is used in such an environment.			
Test frequency (MHz)	Band a) (MHz)	Service a)	Modulation b)
385	380 -390	TETRA 400	Pulse modulation b) 18 Hz
450	430 - 470	GMR5 460, FRS 460	FM c) 35 kHz deviation 1 kHz sine
710	704 - 787	LTE Band 13, 17	Pulse modulation b) 217 Hz
745			
780			
810			
870	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18 Hz
930			
1720			
1845			
1970	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation b) 217 Hz
2450			
5240			
5500			
5785	5100 - 5800	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation b) 217 Hz
5240			
5500			
5785			
5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation b) 217 Hz
5240			
5500			
5785			

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NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

a. For some services, only the uplink frequencies are included.  
b. The carrier shall be modulated using a 50 % duty cycle square wave signal.  
c. As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

# C CARE

## **WEEE Directive & Product Disposal**

At the end of its serviceable life, this product should not be treated as household or general waste. It should be handed over to the applicable collection point for the recycling of electrical and electronic equipment, or returned to the supplier for disposal.

## **Internal / Supplied Batteries**

The battery is to be collected separately.

The battery is designed for separate collection at an appropriate collection point.

# C CARE

CRESTA INTERNATIONAL B.V.

Rolbrugweg 4 - 1332 AS - Almere - The Netherlands

Info@cresta-international.com - + 31 20 6 932 932

**[www.cresta-international.com](http://www.cresta-international.com)**