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EN Additional Information Condoms: Handle carefully, when removing from package avoid damage by fingernails, jewellery etc. If being damaged replace an individual contraceptive condom as soon as possible. Do not use a condom pharmacist about compatibility of any topical medicines possible contact with condom. If additional lubricant should be added, use only those recommended for condoms and never oily ones like petroleum jelly, baby oil, body lotion, etc. Apply lubricant to the outside of the condom, penis before and contact between penis and partner body areas to assist preventing STIs or pregnancy. Stop if you feel condom slipping off or tightening excessively onto penis because of breakage danger. Withdraw penis soon after ejaculation. Do not use a condom if it has been used. Should a condom burst or leak during use seek medical assistance as soon as possible, at least within 72 hours. Dispose condom to waste, not to toilet. Don't use twice, reuse might increase risk of failure or infection. For vaginal use use a new condom every time. Condoms are not perfect. Noisy sex, huzzy condoms are a reliable protection but no

contraceptive can protect 100 % against pregnancy, AIDS and/or infectious diseases.

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