**VULPES GOODS**

BABYCARE

MANUAL

<b>ENGLISH</b>	<b>3</b>
The Device	4
User guide	7
Characteristics and Application	8
Product Parameters	11
<b>DEUTSCH</b>	<b>12</b>
Das Gerät	13
Benutzerhandbuc	16
Eigenschaften und Anwendung	17
Produkt parameter	20
<b>NEDERLANDS</b>	<b>21</b>
Het apparaat	22
Gebruiksaanwijzing	25
Kenmerk en toepassing	26
Product Parameters	29
<b>FRANCAIS</b>	<b>30</b>
Le dispositif	31
Manuel de l'utilisateur	34
Caractéristiques et applications	35
Paramètres du produit	38

**VULPES GOODS**

BABYCARE

**ENGLISH**

# THE DEVICE

Suction head  
Collection cup

Led Screen  
On/Off button  
Gear button  
Light button  
Music button

Suction heads

Tweezers

charging cable

## **Suction head**

You can use the 3 different attachments for different blockages. For example, deep mucus, shallow mucus and hard pieces.

## **Collection cup**

The collection tray collects all residues. Clean the collection tray on time. The nasal cleaner will not work without a collection tray.

## **Led screen**

Use the LED screen to read the mode and battery status.

## **On/Off button**

Press once to turn the electric nasal aspirator on, then press again to turn it off.

## **Gear button**

Pressing the button selects one of the five available suction levels. Each click increases the suction power. After suction level 5, the suction level goes back to 1.

## **Light button**

Press the light button once to turn on the light and illuminate the nasal cavity. Press again to turn it off.

## **Music button**

If you press the music button, 3 different songs can be played. Each time you press the button, the song changes. After the third song, it will go back to mute.

## **Tweezers**

Tweezers for extra cleaning of your product.

## **Charging cable**

Use the charging cable to charge the product when needed. Only use the right adapter when charging.

## **USER GUIDE**

Before using the nasal cleaner for the first time, it is important to make sure the cleaner is fully charged for optimal performance.

1. Place the collection cup on the nasal cleaner and choose the appropriate attachment
2. Press the power button to activate the nasal cleaner and choose the desired suction strength.
3. For an optimal experience, you can use the music button and choose from 3 different children's songs.
4. For an even better experience, you can also use the light button. This is also possible while using the music button.
5. After using the nasal cleaner, clean the used attachments and collection cup with lukewarm water.

## **CHARACTERISTICS AND APPLICATION**

- Do not expose this product to high temperature, high humidity or direct sunlight for a long time.
- Prevent the occurrence of intense pressure collisions or accidental drops of this product.
- This product is only suitable for absorbing nasal secretions. Do not use it for improper use, such as absorbing water or skin.
- When the suction is significantly reduced or the motor sound is significantly reduced, please charge it in time.
- When using this product, make sure the user does not hold their breath or swallow saliva. Keep the mouth open to avoid pressure differences in the ear.
- In Case of any pain or discomfort during use. Stop immediately, and seek medical attention!



- Avoid inserting the suction head into the nasal cavity with excessive pressure.
- Clean the suction head before each use and keep it clean.
- Adults must adhere to the operating instructions when using it, and it is prohibited for children to operate it independently. Ensure the product is stored in a location inaccessible to children.
- Not possible to put your own music on the product.
- Do not put product in dishwasher and do not submerge in water or liquid.

- Some weak slime can flow back (If you are able to clear the suction port of the main engine using a cotton swab).

- Utilize this product in the correct posture/method to prevent liquid from being drawn into the main device.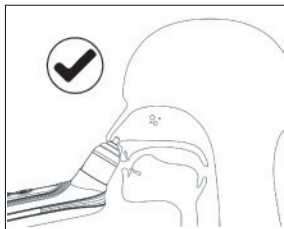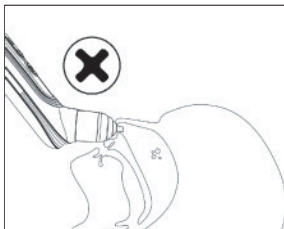

## **PRODUCT PARAMETERS**

Productname:	Electric nasal aspirator PRO
Model:	VG-UB-007
Battery:	450MA
Rated input voltage:	5V
Rated input power:	5W-1A

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