Read Instruction manual (1) and (2) before use. For symbols information, refer to "Symbols Description" on the back side of the another "Instruction manual (1)".

Introduction Thank you for purchasing the M2 Intelli IT Automatic Upper Arm Blood Pressure Monitor. This blood pressure monitor uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

Safety Instructions This instruction manual provides you with important information about the M2 Intelli IT Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, PLEASE UNDERSTAND all of these instructions. If you do not understand these instructions or have any questions, contact your OMRON retailer or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.

Intended Use This device is a digital monitor intended for use in measuring blood pressure and pulse rate in a healthy adult population. This monitor detects the appearance of irregular heartbeats during measurement and indicates this by a symbol with readings. It is mainly designed for general household use.

Receiving and Inspection Remove this monitor from the packaging and inspect for damage. If this monitor is damaged, DO NOT use and consult with your OMRON retailer outlet or distributor.

Important Safety Information Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your use. Keep for future reference. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. DO NOT use this monitor on infants, toddlers, children or adults who cannot understand or follow instructions. DO NOT use this monitor on people who are unconscious or who are unable to cooperate with the measurement. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied.

Caution Indicates a potentially hazardous situation which, if not avoided, may result in moderate injury to the user or patient, or cause damage to the equipment or other property. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied.

Information This product emits radio frequencies (RF) in the 2.4 GHz band. DO NOT use this monitor in areas containing high frequency RF such as in hospitals, turn off the Bluetooth® feature in this monitor, remove batteries and/or unplug the AC adapter when in RF sensitive areas. DO NOT use this monitor in areas containing high frequency RF such as in hospitals, turn off the Bluetooth® feature in this monitor, remove batteries and/or unplug the AC adapter when in RF sensitive areas.

General Precautions When you take a measurement on the right arm, the rest your arm on the air tube. When you take a measurement on the left arm, the rest your arm on the air tube. Blood pressure may differ between the right and left arm, and may result in a different measurement value. Always use the same arm for measurement. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied. DO NOT use this monitor on people who are wearing any clothing, jewelry, or accessories on the arm where the cuff is applied.

Table with 3 columns: Display/Problem, Possible Cause, Solution. Problems include ES appears, ES does not flash, ES appears with a red symbol, ES appears with a red symbol, ES appears with a red symbol, ES appears with a red symbol.

More than 24 readings are not transferred. Your monitor is not paired or not connected with your smart device. There are 30 readings in memory to be transferred. Batteries are low. Batteries are depleted. Batteries are being recharged. Nothing appears on the display of the monitor.

Readings appear too high or too low. Blood pressure values constantly. Many factors including stress, time of day, and/or how you apply the arm cuff may affect your blood pressure. Follow the instructions shown in the smart device app. The OMRON connect app for further help. If the problem still persists, contact your OMRON retailer outlet or distributor.

2. Maintenance 2.1 Maintenance To protect your monitor from damage, follow the directions below: - Changes or modifications not approved by the manufacturer will void the warranty. 2.2 Storage Keep your monitor in the storage case when not in use.

1. Error Messages and Troubleshooting If any of the below problems occur during measurement, check to make sure that the monitor is used in accordance with the manual. If the problem persists, please refer to the table below.

Table with 3 columns: Display/Problem, Possible Cause, Solution. Problems include E1 appears or inflates, E2 appears or inflates, E3 appears, E4 appears.

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4. Limited Warranty Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing. It is designed to give you a long, useful life.

5. Guidance and Manufacturer's Declaration This blood pressure monitor is designed according to the European standard EN1060. Non-invasive sphygmomanometry Part 1: General Requirements and Part 3: Supplementary requirements for electromagnetic blood pressure measuring systems.

Correct Disposal of This Product (Waste Electrical & Electronic Equipment) This marking shown on the product or its literature, indicates that it should not be disposed of with other household wastes. To prevent possible harm to the environment and to protect human health from uncontrolled waste disposal, please separate this product from other types of waste and recycle it according to the instructions on the sustainable reuse of material resources.

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M2 Intelli IT (HEM-7143T1-EBK) (HEM-7143T1-EBK) أم 2 انتلي اي تي

All for Healthcare

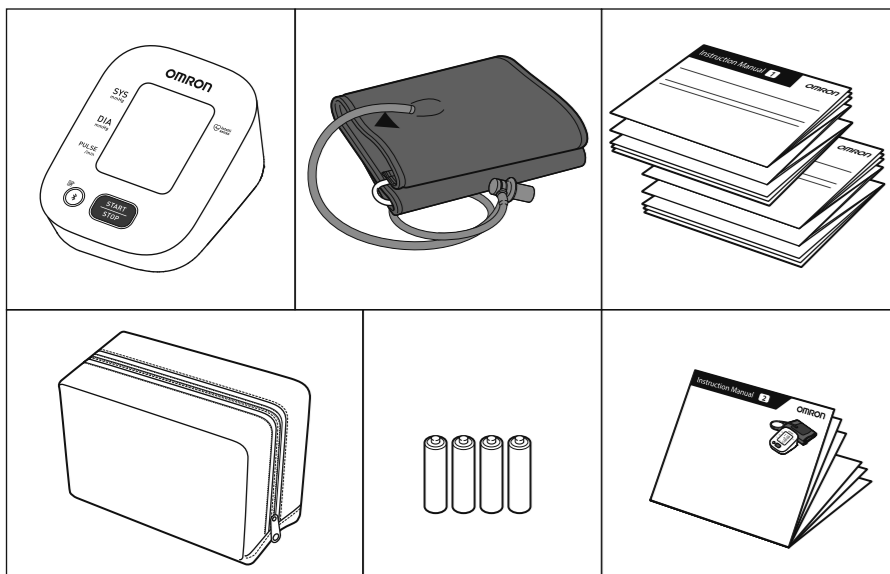
Read Instruction manual 1 and 2 before use.

- FR Lire le mode d'emploi 1 et 2 avant l'utilisation.
- DE Lesen Sie vor der Verwendung Gebrauchsanweisung 1 und 2.
- Italiano Leggere il manuale di istruzioni 1 e 2 prima dell'uso.
- Español Lea el manual de instrucciones 1 y 2 antes del uso.
- Nederlands Lees gebruiksaanwijzing 1 en 2 voor gebruik.
- Русский Перед использованием прочтите руководства по эксплуатации 1 и 2.
- Türkçe Kullanmadan önce kullanım kılavuzu 1 ve 2'yi okuyun.

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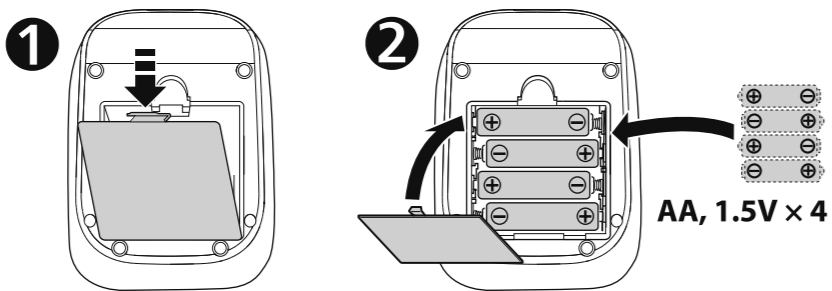
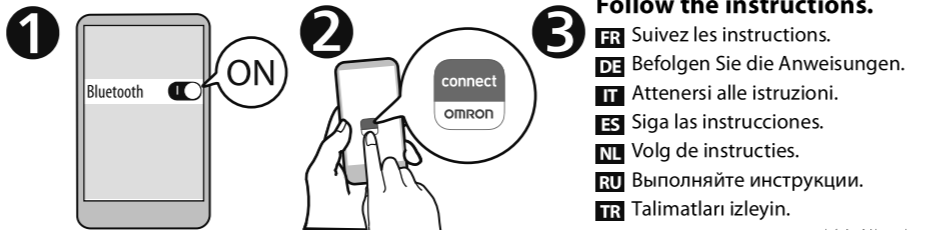
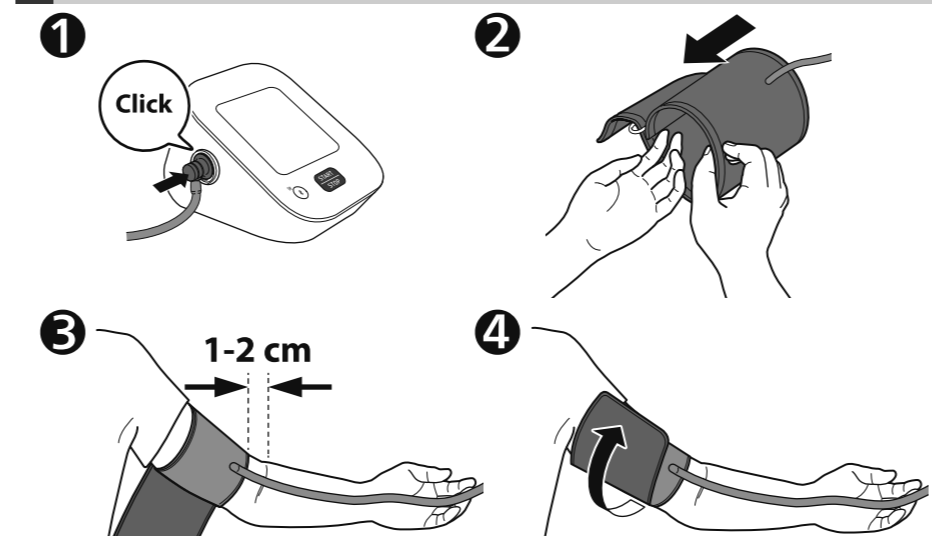
3 Tube side of the cuff should be 1 - 2 cm above the inside elbow.

- FR Le côté tuyau du brassard doit être positionné 1 à 2 cm au-dessus de l'intérieur du coude.
- DE Das Manschettenstück mit dem Schlauch muss 1 bis 2 cm oberhalb des Ellbogens liegen.
- IT Il lato del bracciale con il tubo deve trovarsi al di sopra del gomito interno, a una distanza di circa 1 o 2 cm.
- ES El lado del tubo del manguito deberá quedar 1 o 2 cm por encima de la parte interna del codo.
- NL De kant met de slang van de manchet moet 1 - 2 cm boven de binnenkant van de elleboog liggen.
- RU Край манжеты с трубкой должен находиться на 1-2 см выше локтевого сгиба.
- TR Koluğün boru tarafı, dirsek içinin 1 ila 2 cm üstünde olmalıdır.

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4 Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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If taking measurements on the right arm, refer to:

- FR Pour la prise de mesures au bras droit, voir :
- DE Bei Messungen am rechten Arm siehe:
- IT Se la misurazione viene eseguita al braccio destro, fare riferimento a:
- ES Si va a realizar mediciones en el brazo derecho, consulte:
- NL Als u metingen aan de rechterarm uitvoert, raadpleeg dan:
- RU При измерении на правой руке см.
- TR Sağ koldan ölçüm yapıyorsanız aşağıdakilere bakın:

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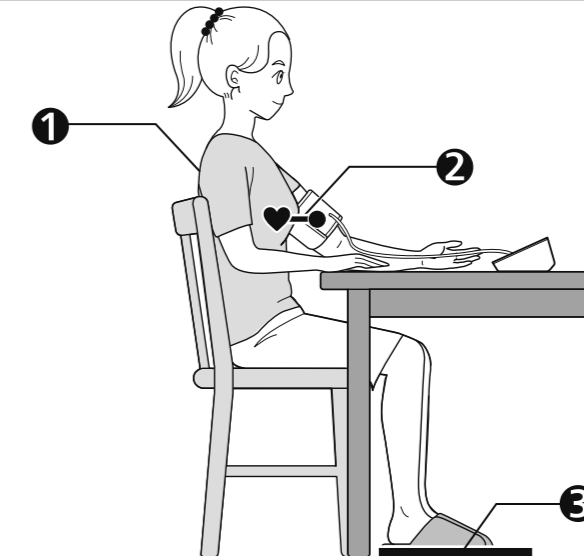
1 Sit comfortably with your back and arm supported.

- FR S'asseoir confortablement de manière à ce que le dos et le bras soient bien soutenus.
- DE Bequem hinsetzen, mit dem Rücken anlehnen und den Arm auf eine Unterlage legen.
- IT Sedere comodamente con la schiena e il braccio ben sostenuti.
- ES Siéntese cómodamente de modo que la espalda y el brazo estén bien apoyados.
- NL Neem een comfortabele zitpositie in, met uw rug en arm ondersteund.
- RU Сядьте удобно, чтобы спина и рука опирались на что-либо.
- TR Sirtınız ve kolunuz desteklenerek rahat bir şekilde oturun.

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2 Place the arm cuff at the same level as your heart.

- FR Le brassard doit se trouver au même niveau que votre cœur.
- DE Die Manschette auf Herzhöhe platzieren.
- IT Posizionare il bracciale allo stesso livello del cuore.
- ES Coloque el manguito al mismo nivel que el corazón.
- NL Plaats de armmanchet op hetzelfde niveau als uw hart.
- RU Манжета должна находиться на уровне сердца.
- TR Koluğün kalbinizle aynı düzeyde getirin.

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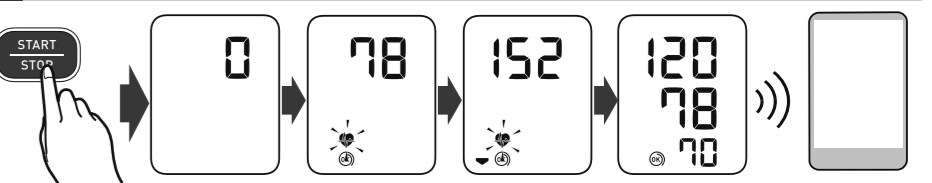
3 Keep feet flat, legs uncrossed, remain still and do not talk.

- FR Garder les pieds à plat, les jambes non croisées, ne pas bouger et ne pas parler.
- DE FüÙe flach auf den Boden stellen, die Beine nicht kreuzen, still halten und nicht sprechen.
- IT Tenere i piedi ben poggianti, non incrociare le gambe, rimanere fermi e non parlare.
- ES Mantenga los pies planos, las piernas sin cruzar, quédese quieto y no hable.
- NL Hou de voeten plat, benen niet gekruist, zit stil en praat niet.
- RU Поставьте ноги ровно, не перекрещивая их, не двигайтесь и не разговаривайте.
- TR Ayaklarınızı düz, bacaklarınızı açık tutun, hareketsiz kalın ve konuşmayın.

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When the [START/STOP] button is pressed, the measurement is taken and saved automatically. Open the app to transfer the reading.

- FR Après une pression sur le bouton [START/STOP], la mesure démarre et le résultat est enregistré automatiquement. Ouvrir l'application pour transférer le résultat.
- DE Beim Drücken der Taste [START/STOP] erfolgt die Messung. Die Messwerte werden automatisch gespeichert. Öffnen Sie die App, um die Messwerte zu übertragen.
- IT Quando si preme il pulsante [START/STOP], la misurazione viene eseguita e salvata automaticamente. Aprire la app per trasferire i risultati.
- ES Al pulsar el botón [START/STOP], se obtiene la medición, que se guarda automáticamente. Abra la aplicación para transferir la lectura.
- NL Wanneer op de knop [START/STOP] wordt gedrukt, wordt de meting automatisch uitgevoerd en opgeslagen. Open de app om de meting te versturen.
- RU После того, как кнопка [START/STOP] нажата, измерение выполняется и сохраняется автоматически. Откройте приложение, чтобы передать результаты.
- TR [START/STOP] düğmesine basıldığında ölçüm yapılır ve otomatik olarak kaydedilir. Ölçüm değerini aktarmak için uygulamayı açın.

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If your systolic pressure is more than 210 mmHg: After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

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