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Off key after the measurement. Otherwise the thermometer will automati- cally switch off after 10 minutes.

**Memory**

The last measured value will be stored automatically. This value is displayed when the On/Off button is pressed. The memory symbol is also shown beside the measured value, e.g. 37.5°C.M.

**Changing the battery**

The clinical thermometer contains a long-life alkali-manganese battery (LR 41, 1.55 V), which will last for approx. 2 years. The battery has to be replaced when "T" appears in the bottom right corner of the display. Press down the battery cover at the end of the device with your fingers and then pull it firmly back. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then put the battery cover back onto the case. Used batteries do not belong in the regular domestic refuse. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries.

Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain mercury (Hg) and lead (Pb).

**Cleaning/Disinfection**

The thermometer's flexible tip as well as its display are waterproof. This part of the thermometer can easily be submerged in water or disinfectant solution during cleaning or disinfection.

Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.

**Name of the agent:** Isopropanol 70% Pharmacy; Gigasept FF; Lysoformin; Aseptisol.

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<b>Table 4</b>			
<b>Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT or ME SYSTEM</b>			
The BY 11 is intended for use in an electromagnetic environment which radiated RF disturbances are controlled. The customer or the user of the BY 11 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BY 11 as recommended below, according to the maximum output power of the communications equipment.			
<b>Rated maximum output power of transmitter</b>	<b>Separation distance according to frequency of transmitter (m)</b>		
	<b>150 kHz to 80 MHz</b>	<b>80 MHz to 800 MHz</b>	<b>800 MHz to 2,5 GHz</b>
	<b>d = 1,2 <math>\sqrt{P}</math></b>	<b>d = 1,2 <math>\sqrt{P}</math></b>	<b>d = 2,3 <math>\sqrt{P}</math></b>
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance <i>d</i> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. <p><b>NOTE 1</b> At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. <b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

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**Storage/Disposal**

When you are not using the thermometer, please retain it in the original protective cover.

The accuracy of this thermometer has been carefully checked and developed with regard to a long useful life.

If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address. Please dispose of the device in accordance with the directive – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

**Technical data**

Model:	BY 11
Type:	Maximum thermometer type FT 15/1
Range of measurement:	32°C to 42.9°C
Accuracy of measure- ment:	±0.1°C in a water bath with a temperature of 35.5°C to 42.0°C <p>±0.2°C in a water bath with a temperature of 32.0°C to 35.4°C</p> ±0.2°C in a water bath with a temperature of 42.1°C to 42.9°C
Ambient temperature:	10°C to 45°C, with 30% to 95% relative humidity
Storage temperature:	-10°C to +60°C, with 25% to 95% relative humidity

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