Spare Teats

Ideal for feeding breast milk

Note: Only use Medela original accessories.

Make sure breastfeeding is well established before introducing a teat.

Cleaning and Hygiene

- | Use drinking water quality for any of the following described cleaning procedures.
- | Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- | Take care not to damage parts when using a brush. Inspect all items before each use.
- | Replace parts if damaged or worn out.
- | For hygiene reasons, we recommend replacing teats after 3 months.
- | Note: Food colors may discolor components which will not affect performance.
- | Follow these cleaning instructions unless you are told otherwise by your healthcare professional.
- | Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant (*sterilizing solution*) for longer than recommended, as this may weaken the teat.

Prior to first use and once daily

- | Disassemble, wash and rinse all parts.
- | Place parts in a saucpan of sufficient capacity.
- | Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of lime scale.
- | Heat it up and allow to boil for 5 minutes.
- | Remove from stove and drain water off.
- | Dry parts with a clean towel.

As an alternative to boiling:

Dishwasher: Clean parts in top rack.

Microwave: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.

Steam sterilizer: Follow the manufacturer's instructions.

After each use

- | Disassemble and wash all parts in warm, soapy water.
- | Rinse in clear water. Dry parts with a clean towel.
- | After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- | Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- | Do not place in direct sunlight.
- | Avoid touching the inside of bottles and lids.

For your child's safety and health

WARNING!

- | Always use this product with adult supervision.
- | Never use feeding teats as a soother.
- | Continuous and prolonged sucking of fluids will cause tooth decay.
- | Always check food temperature before feeding
- | Keep all components not in use out of the reach of children.
- | Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

Material: Silicone

Disposal: According to local regulations
Conforms to EN 14350.

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