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NasoFree® nasal irrigator

The NasoFree® Nasal Irrigator and Dos Medical Nasal Saline™ is used to rinse the nasal cavity and paranasal sinuses

- in the case of acute and chronic sinusitis (= rhinosinusitis)
 - in the case of respiratory allergy such as grass and tree pollen (hay fever), house dust mite and animal dander such as cat, dog, horse.
 - during the recovery after nasal surgery, among other things, to prevent crusting.
 - in case of flu or common cold
 - when rinsing the nasal cavity as a result of exposure dust and polluted air
- If snoring is caused by a reduced passage of the nasal respiratory tract, the NasoFree® nasal irrigator can help to prevent snoring.*

Only use your own NasoFree® nasal irrigator.

To avoid contamination, each family member should use his own NasoFree® nasal irrigator. NasoFree® is a Class I medical device that you use on a daily basis. For reasons of wear and hygiene, it is recommended to replace the device after one year for a new NasoFree® nasal irrigator.

Important

- Ask your doctor if your symptoms do not improve.
- **BE CAREFUL not to rinse with (too) hot rinsing solution and never block the nostril through which the nasal saline must flow outwards.**
- Only use NasoFree® when standing or sitting.
- Always rinse your nose with nasal rinse salt. Rinsing with normal water is painful and can cause irritation. Rinsing with nasal saline cleans better than rinsing with normal water.

Nasal rinsing and medication

If you use a nasal spray, rinse your nose before applying the nasal spray. Your medication then end upon clean nasal mucosa. After you have rinsed your nose, droplets may flow back from your nose. Therefore wait a few minutes before applying your nasal medication. After nasal rinsing, you can also improve your nasal mucosa with NasuMel® Nasal Ointment (nasumel.nl).

How often can you rinse?

Depending on the degree of your symptoms, you can rinse your nose as needed. Below you will find a guideline, but always follow the advice of your doctor.

- After exposure to dust or polluted air: 1 x per day.
- Preventive in case of respiratory allergies, mild nasal blockage or discomfort: 1 - 2 times a day.
- In case of post nasal/ sinus surgery, moderate / severe nasal blockage, crusts or infection: 2 - 4 times a day.

Instruction for use NasoFree® irrigation

(For a video tutorial, please see www.nasofree.nl)

1. Empty the nasal saline sachet into the NasoFree and fill it with lukewarm (drinking)water (37 °C max.). Shake the NasoFree® a little before use. Be sure to have a handkerchief , towel or tissue during nasal rinsing to wipe your nose during nasal rinsing.
2. Bend slightly forward and place the nasal irrigator at an angle (of about 45°) in one of your nostrils.

3. Gently squeeze the NasoFree® and let the nasal saline solution flow out of your nose after squeezing. The nasal saline can flow out through the same nostril (in case of a severely nasal blockage) and through the other nostril. Both are good. If necessary, blow your nose lightly. Place the tip of the NasoFree® in the other nostril and repeat this procedure until the nasal irrigator is empty. By gently squeezing, you prevent the nasal saline from entering your pharynx. Should this happen, it is not harmful and you just spit it out.

When should you not rinse with the NasoFree®?

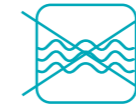
- If there is a direct connection to the fluid cavities of the brain due to an injury to the nasal fossa and paranasal sinuses.
- In case of heavy nosebleed.

I sometimes hear my ears 'crack' when I muzzle the mucus out of my nose after nasal rinsing

When you blow your nose after nasal rinsing, pressure differences develop in the nasal cavity causing your 'ears to burst'. This can not hurt. It is similar to simply blowing your nose in a handkerchief.

Cleaning the NasoFree® nasal irrigator

After use, clean the (parts of) the NasoFree® by rinsing it clean (if necessary using a drop of detergent) under running, warm water. Disinfect the nasal irrigator regularly, eg once every 1 - 2 day (s). This can be done by placing the parts of the nasal irrigator in a pan / large bowl and pouring / dipping it with boiling water. After 10 seconds, take out the parts and let them air dry. Do not boil the nasal douche. Too long in boiling water can deform the nasal irrigator. Allow the nasal douche parts to air dry after cleaning. Do not screw the screw cap onto the nasal irrigator during the drying phase.

Dos Medical Nasal Saline™

Dos Medical Nasal Saline™ is specially composed for rinsing of the nasal cavity and paranasal sinuses. It contains no iodine, as iodine irritates. The combination of sodium chloride and sodium bicarbonate effectively cleans and protects the sensitive nasal mucosa. One sachet contains just the right amount for a nasal rinse containing 250 ml (isotonic solution). Dissolve 1 sachet of nasal saline in 250 ml of lukewarm (drinking)water (maximum 37 ° C). Lukewarm water is more comfortable to rinse your nose than cold water. Isotonic nasal saline solutions can generally be used without limitation in duration and frequency.

Dos Medical Nasal Saline with Xylitol

is specially developed for people with chronic sinusitis (chronic rhinosinusitis). In chronic rhinosinusitis bacteria often play a role, forming a "biofilm" (= a kind of mucus layer). Because of this mucus layer, bacteria attach better to tissue and the effect of an antibiotic is undermined. Scientific research has shown that xylitol breaks down this biofilm and that the cleansing of the paranasal sinuses in people with chronic nasal cavity inflammation with the combination of nasal saline with xylitol gives a better result than nasal rinse with nasal saline alone.

dosmedical
dedicated to healthcare

 **DOS Medical B.V.**
Laan van Westenenk 64
NL-7336 AZ Apeldoorn

Tel. +31 (0)85 773 20 80
informatie@dosmedical.nl
www.dosmedical.nl

NF-FU-001

dosmedical
dedicated to healthcare

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