

Mission® Capillary Transfer Tubes

1. When using a capillary tube, it is important not to squeeze the bulb. Holding the tube slightly downward, touch the tip of the capillary tube to the blood drop. The blood will automatically be drawn in to the black fill line and stop.
2. Make sure that blood covers the black line and the air vent, or it will be difficult to squeeze the blood out.
3. Align the tip of the capillary transfer tube with the Specimen Application Area on the test strip. Squeeze the bulb to apply the blood sample to the Specimen Application Area.

REF C121-3081

LOT 1546/2000

2020-06

50

SIZE 35µl

ACON®
ACON Laboratories, Inc.
10125 Mesa Rim Road,
San Diego, CA 92121, USA

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

