

 **Okped** TECHNICAL SHOES  
**orione**<sup>®</sup>

MEDICAL DEVICE CLASS I ACCORDING TO COUNCIL DIRECTIVE 93/42  
EEC AND UPDATE DIRECTIVE 2007/47/EC

**WARNING**

Consult the physician and/or the orthopaedic technician for appropriate time and way of use.

Avoid direct contact with injured skin.

Do not use together with ointments and creams.

It's not a sterile device.

The numbering corresponds to the one commonly used for normal footwear.

**INDICATIONS**

Are not corrective or curative footwear.

Their use is subject to the opinion of the doctor.

Help walking if case different pathologies do not allow the use of normal shoes (post-operative, post-trauma, swelling of the foot, etc).

Do not change shoe structure.

**CARE AND CLEANING**

Wash with a damp cloth and neutral soap. Avoid direct immersion in water.

Dry away from heat sources

**DISPOSAL**

Dispose in accordance with the regulations established by current legislation



SAFTE S.p.A. - Via Adamello 13  
10093 - Collegno (TO) Italy  
Tel. +39 011.4038111 - Fax +39 011.4038126  
e-mail: info@safte.it www.safte.it

