BRAUN

Nasal aspirator 1

- ⚠

Please read all the instructions before using the Braun Nasal aspirator, especially this basic safety information.

- The nasal aspirator is designed to clean/extract nasal secretions only. Do not use the device for purposes other than those stated in these instructions for use.

- Keep out of reach of children under 12 years.

- Do not expose the Braun Nasal aspirator to direct sunlight.

- This device is designed for domestic use only. Consult a doctor if the child is suffering from a nasal infection.

- Do not use other nozzle than those supplied with this device.

Guarantee

Consumer Card available on our website at www.hot-europe.com/support

Please see last page of this manual to find the contact for the Kaz Authorized Service Center in your country.

UK only: This guarantee in no way affects your rights under statutory law.

LOT and Serial numbers of your product must be provided as they are essential to record and follow up your inquiry or claim.

The LOT and Serial numbers are located in the battery compartment.

The production date is given by the LOT number and can be deciphered as explained below:

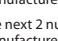
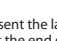
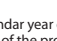
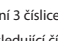
The first 3 numeric digits in the LOT number represent the day of the year of manufacture.

The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letter(s) at the end designate the manufacturer of the product.

(E.g.: LOT No.: 12313tav: this product was made on the day 123, year 2013 at the manufacturer code tav.)

© 2018, All rights reserved.

BNA100EU P/N A002276R0 20JUN18

		
CE 0297	PAP	22
		
Printed in China		

Specifications	Model	Model	Model
Maximum suction power	BNA100	BNA100	BNA100
Operating temperature	70 kPa	70 kPa	70 kPa
Storage temperature	+10 °C ~ +40 °C	+10 °C ~ +40 °C	+10 °C ~ +40 °C
Operation & storage relative humidity	-25 °C ~ +55 °C	-25 °C ~ +55 °C	-25 °C ~ +55 °C
Unit weight	≤95 % R.H. non-condensing	≤95 % nekondenzující	≤95 % nekondenzující
Power source	230 g	230 g	230 g
Battery life	Alkaline battery: 2 x AA (LR6) 1,5V	alkalická baterie 2 x AA (LR6) 1,5V	alkalická baterie 2 x AA (LR6) 1,5V
Service life	1000 operations - 3 hours	1000 použití - 3 hodiny	1000 betjeninger - 3 timer
Warranty	5 years	5 let	5 år
	2 years	2 roky	2 år

			
Equipment with type BF applied parts	See instruction for use	Operating temperature	Storage temperature

Subject to change without notice.

IP22 Protected against solid foreign objects of 12.5 mm diameter and larger. Protected against water drops falling vertically when the device is tilted up to 15°.

This product conforms to the provisions of the EC Directive 93/42/EEC (Medical Device Directive and superseding regulations).

This product carries the CE mark and is manufactured in conformity with the RoHS Directive 2011/65/EU.

The Braun Nasal aspirator needs special precautions regarding EMC.

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- ⚠

This product contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in household waste, but take it to an appropriate local collection centre provided in your country.

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LOT and Serial numbers of your product must be provided as they are essential to record and follow up your inquiry or claim.

The LOT and Serial numbers are located in the battery compartment. The production date is given by the LOT number and can be deciphered as explained below:

The first 3 numeric digits in the LOT number represent the day of the year of manufacture.

The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letter(s) at the end designate the manufacturer of the product.

(E.g.: LOT No.: 12313tav: this product was made on the day 123, year 2013 at the manufacturer code tav.)

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