User Manual

Y-201

Smart Ear Wax


MADE IN CHINA

1 Scan the QR code .QR


2 Select an available APP store to install the APP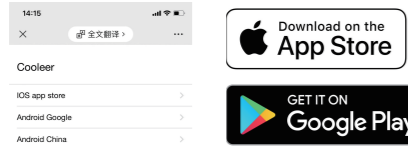


EN

3 Open the APP to add the device "Smart Ear Wax Removal Kit"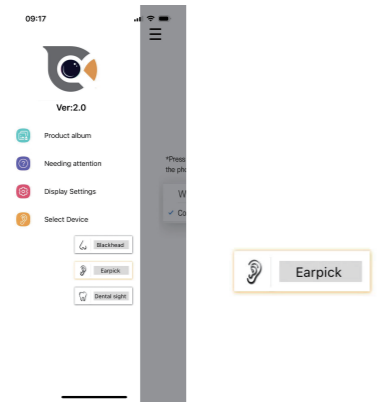


4 Turn on the smart ear wax removal to complete the connection

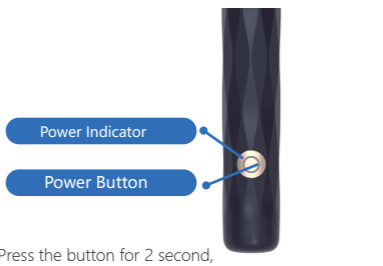Power Indicator
Power Button

Press the button for 2 second, the indicator will flash and the lens light will light on.


5 Connect the smartphone to the product's WIFI

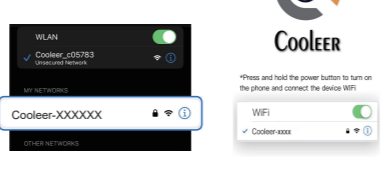Enter the APP interface, and click the button to switch to working mode.

Functions of APP Buttons

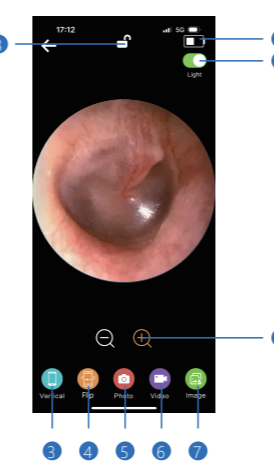- 1 Lens light
- 2 Screen rotation
- 3 Take photo
- 4 Album
- 5 Remaining Battery
- 6 Zoom in and out
- 7 Mirror the image
- 8 Take video
- 9 Screen lock

Precautions

- The temperature of the product will rise slightly when used; the maximum will not exceed 40°C, it has no harm to the human body, please use it confidently.
- It is recommended to wipe the lens gently with an alcohol swab when clearing.
- Please observe the surrounding area before use to ensure the environment is safe.
- Please do NOT use this product for children under 3 years old.
- Minors should NOT use this product alone to avoid accidental injury.
- Do NOT expose the product body to liquids to avoid water damage.
- The product has a built-in lithium battery, so please charge it once a month to maintain the battery life when long time no use.

Product Parameters

Product Name:	Smart Ear Wax Removal Kit
Model Number:	Y-201
Web Standards:	802.11g/n protocol, 802.11b protocol
Antenna Type:	Internal antenna
Working Frequency:	2.4Ghz
Image Transfer Rate:	20-30fps
Image Sensor:	CMOS
Pixel:	3 megapixels
Battery Capacity:	300 mAh
Working Time:	90 min
Charging Time:	60 min
Input Current:	USB 5V 0.5A
Lens Diameter:	3.5 mm
Best Focal Length:	15-20 mm
Depth of Field:	10-50 mm
Gyro:	5 axis
Working Temperature:	-10-45°C

Common Problems and Solutions

Problems	Solutions
The camera is blurry/unclear.	Wipe the lens gently with an alcohol swab.
No response when booting.	Try again after fully charging.
The App crashes.	Clean up the running APPs of the phone, and then restart the APP.
The camera light does not light on.	Try restarting the product.
The product flashes a red light.	Low battery; try again after fully charged.
After the product is successfully connected, the real-time image cannot be displayed normally.	Disconnect the phone from the product's WIFI, restart the product, reopen the APP and reconnect to the product's WIFI.
The WIFI connection is unstable and disconnected after a few seconds.	Restart the product and reconnect to the product's WIFI.

Warm Tip: If the content above has not solved your problems, please contact the seller through order. We will help you out in 24 hours.

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