SAM MAX is a brand that cares. We not only want to provide premium quality first class condoms to you. We also want to support people who need them too but cannot afford them and educate people that are not familiar with condoms. We aim to make the world a safer sexual environment.

For every condom that you buy, we will donate one. The distribution of the donated condoms will be carried out by international aid organisations. Besides donation, we will allocate resources and provide peer to peer education together with our user group. This growing user community can propose and initiate new projects and participate in educational programs.

Gullin HBM Health Protections, Inc.
No.1-2, Shujing East Road
Gullin, China

HBM Medical, Coliemore House,
Coliemore Road, Dalkey,
Co Dublin, Ireland

CE 0344

EN Additional Information Condoms:
1. For contraception and prevention of sexually transmitted infections. 2. Handle carefully, when removing from package avoid damage by fingernails, jewelry etc. 3. Do not use the condom if the rubber material is sticky or brittle or the individual container is obviously damaged. 4. If being damaged replace in individual container by a new one and discard the old one. 5. Ask a doctor or pharmacist about compatibility of any topical medicines in possible contact with condom. 6. If extra lubrication is needed, use water-based lubricants and never only ones like petroleum jelly, baby oil, body lotion, massage oil, butter, margarine etc. 7. Place condom on erect penis before any contact between penis and partner body occurs to assist preventin g STIs or pregnancy. 8. Stop if you feel condom slipping off or tightening excessively onto penis because of breaking danger. 9. Withdraw penis soon after ejaculation by holding condom in place at penis base. 10. Should a condom burst or leak during use seek medical assistance as soon as possible, at least within 72 h. 11. Dispose of used condom into closed rubbish bin, not into toilet. 12. Don't use twice, reuse might increase risk of failure or infection. 13. Normal condoms are for vaginal use only, not anal sex. 14. NR-Latex might cause allergic reaction incl. anaphylactic shock. 15. SAM MAX condoms are a reliable protection but no contraceptive can protect 100 % against pregnancy, AIDS and/or infectious diseases. 16. Store in a cool dry place away from direct sunlight. Keep away from children they can choke or suffocate on uninfilled or broken condom fragments. 17. Use of condom during oral sex increases the risk of asphyxiation or choking. 18. Condom is a medical device to be used by lay persons - adult male or female. 19. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. 20. SAM MAX condoms conform to international standards ISO 4074/EN ISO 4074.

Condoms. Transparent. Flat. Cylindrically formed. Teat-ended. Lubricated. Store in a cool place. Avoid damage. Nominal width: 54mm. For contraception and prevention of sexually transmitted infections.

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