



For the effective and painless removal of medical adhesives, including plasters, bandage, dressings, tape and stoma appliances

- Very mild and non-irritating
- Works directly
- Easy to apply
- Moisturize the skin
- Fresh odor
- Suitable for adults and children

How to use *Limisan*?

1. Spray *Limisan* around the edges and on top of the adhesive.
2. No waiting required. Directly peel the adhesive away from the skin. Spray additional *Limisan* as needed.
3. *Limisan* dries quickly and will not affect the adhesion of subsequent adhesives.

Danger!

Read enclosed leaflet before use. Should not be used if allergic to any of the ingredients. Extremely flammable aerosol. Pressurised container: May burst if heated.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use.

Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.

If medical advice is needed, have product container or label at hand. Keep out of reach of children.

Avoid release to the environment.

Very toxic to aquatic life. Toxic to aquatic life with long lasting effects. Collect spillage. Dispose of contents/container in accordance with local/ regional/ national/ international regulations.



Medisch hulpmiddel
Medical device